



CASE STUDY MINIATURE SCHNAUZER

Clinician: Dr. Diaz
Location: Puerto Rico
Species/Breed: Canis
Breed: Miniature Schnauzer
Age: 11
Gender: Male
Problem: Major wound trauma with infection
Location on patient: Ventral abdomen, abdominal cavity
Product(s) used: Vetericyn Plus[®] VF Antimicrobial Hydrogel, systemic antibiotics



Comments & Notes on treatment: Two days prior to presentation, an 11-year-old male miniature schnauzer named PJ was attacked by two other dogs.

The schnauzer suffered from major wounds to the ventral abdomen. The abdominal cavity had been penetrated, but PJ did not suffer from internal damage. The abdomen was immediately closed with sutures. The skin was too damaged to suture and infection had already begun.

Wounds were surgically debrided and flushed with saline. PJ was placed on systemic antibiotics with daily bandage changing and flushing of wounds.

Three to five days post-surgery, a treatment regimen was implemented using Vetericyn Plus[®] VF Hydrogel as a wound dressing with bandaging. Increased healing and closure of the wound began almost immediately after starting the Vetericyn treatment.



1/10/17



1/12/17



1/13/17



1/16/17



1/27/17



2/1/17

Introducing Vetericyn Plus[®] Veterinary Formula, a new standard in wound care designed to enhance all phases of the wound treatment process.

