



JA is a 36-year-old man who was involved in a motorcycle accident. JA was ejected off of his motorcycle and sustained a concussion and a trauma wound on his left elbow/crease. The wound initially presented with debris from the asphalt and dirt.



INITIAL

Patient presented with a full-thickness wound in the crease and partial thickness on the inner elbow. Treatment began with a 5-minute Puracyn[®] Plus Professional Formula Wound Irrigation Solution soak to both areas, debridement using a Debrimitt[™] to facilitate the removal of the debris, followed by cleansing with Puracyn Plus. Hydrogel was applied to keep wound bed moist, which was covered with a composite dressing. This wound dressing was changed QOD.



DAY 4

On day four there was no longer any need for debridement, which was discontinued. Treatment continued as follows: soak with Puracyn Plus for five minutes, pat dry, apply hydrogel and cover with composite dressing, while still changing the dressing QOD.



DAY 11



DAY 14

CONCLUSION

Both the partial and full-thickness wounds were healed within **14 days**.

LEARN MORE AT PURACYNPLUSPRO.COM